

SmartDataManager

軟件操作說明

The screenshot shows the SmartDataManager application window. At the top left is the 'Ochorder' logo and the title 'Smart Data Manager'. Below the title bar is a search bar labeled 'Search ID/Name' with a magnifying glass icon. To the right of the search bar are two dropdown menus: the first is set to 'MBF-6010' and the second is set to 'COM'. The main area of the window is divided into two sections. The left section is a large, empty white box. The right section contains a list of data fields: 'Data Collection', 'Model', 'S/N', 'Date/Time', 'Body Type', 'Gender', 'Age', 'Height', 'Preset Tare', 'Net Weight', 'BMI', 'BF', and 'FFM'. Each field has a corresponding input field with a small arrow icon. Above the list, there are two tabs: 'Auto' and 'Manual', with 'Manual' currently selected. At the bottom right of the window, there are four buttons: 'Export', 'Collect', 'Print', and 'Save'.

Smart Data Manager

Search ID/Name

MBF-6010 COM

Data Collection Auto Manual

Model

S/N

Date/Time

Body Type

Gender

Age

Height

Preset Tare

Net Weight

BMI

BF

FFM

Export Collect Print Save

目的：

使用此軟件連接 MBF-6010 做傳輸、顯示、紀錄等功能。


架構：



設備：

1. USB Type B 傳輸線
2. 電腦
3. MBF-6010

頁面功能說明：

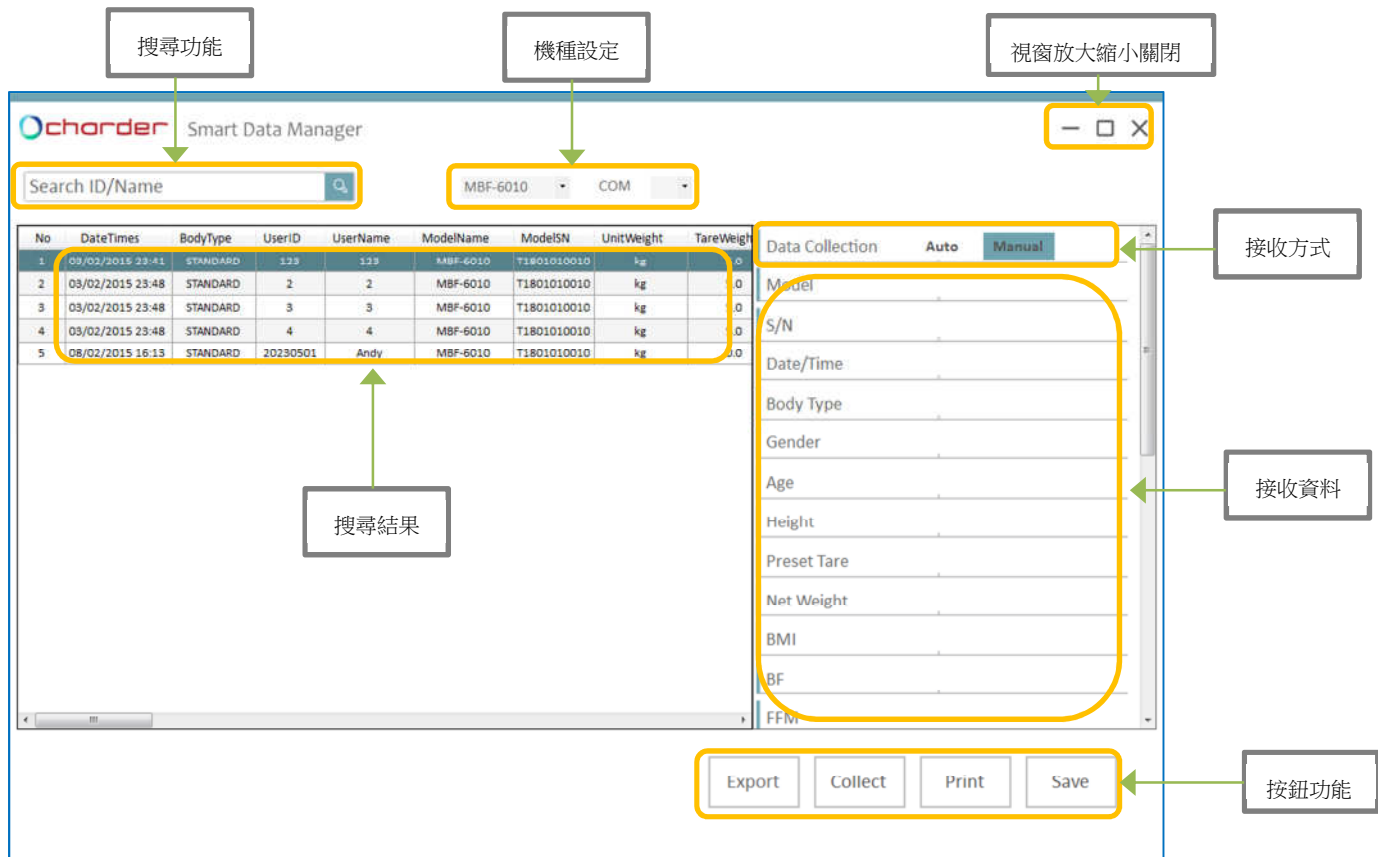

搜尋功能:搜尋文字以 UserID & UserName 相關顯示

機種設定:選擇該機種 & COM 口

接收方式:可選擇 Auto & Manual

接收資料:與 MBF-6010 傳輸內容一致

按鈕功能:

Export:將圖表顯示以 csv 方式另存

Collect:為手動接收資料按鈕

Print:可將當前資料做打印功能

Save:可將當前資料儲存至資料庫

安裝說明：


1. Win10 環境, 須將執行檔點選右鍵內容選擇最高權限人員執行即可

操作說明：


1. 將 MBF-6010 與電腦做 USB 連接
2. 確認已取得 COM 並開啟 SmartDataManager 軟件



3. 選擇要連線機種 MBF-6010 並選擇該 COM 自動連接

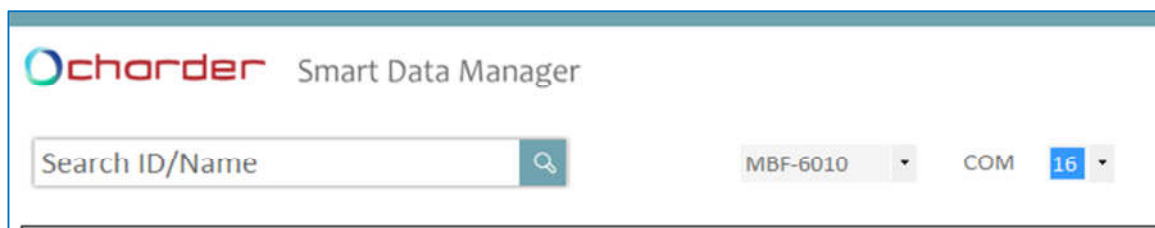

4. 完成連接後,至 MBF-6010 量測, Data Collection 選擇 Auto 自動取得資料或 Manual 手動點選按鈕[Collect]取得資料至 SmartDataManager 軟件

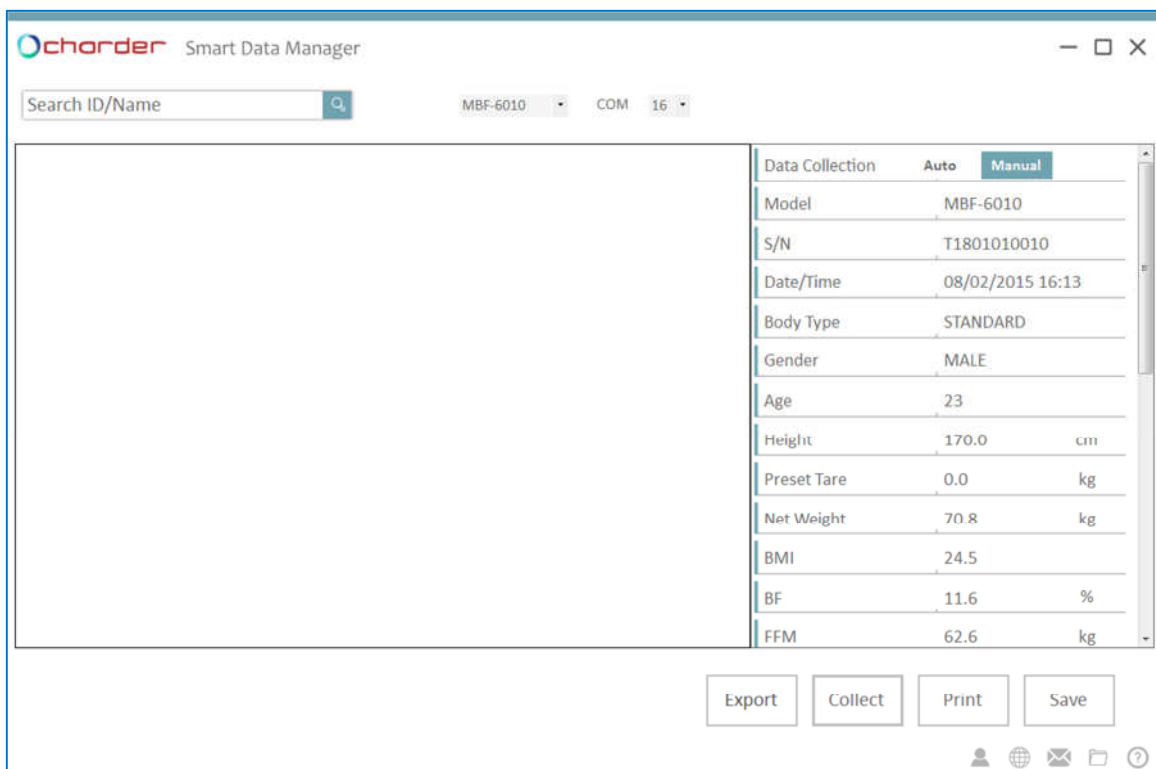

5. 點選按鈕[Save]儲存資料,填入 ID、NAME、身高、生日、性別,完成後自動儲存

Chorder Smart Data Manager

Search ID/Name

MBF-6010 COM 16

Data Collection Auto Manual

Model MBF-6010

S/N T1801010010

Date/Time 08/02/2015 16:13

Body Type STANDARD

Gender MALE

Age 23

Height 170.0 cm

Preset Tare 0.0 kg

Net Weight 70.8 kg

BMI 24.5

BF 11.6 %

FFM 62.6 kg

Export Collect Print Save

6. 在搜尋欄輸入 ID 或 NAME 搜尋測試資料

Chorder Smart Data Manager

Search ID/Name

MBF-6010 COM 16

No	DateTimes	BodyType	UserID	UserName	ModelName	ModelSN	UnitWeight	TareWeight
1	09/02/2015 23:41	STANDARD	123	123	MBF-6010	T1801010010	kg	5.0
2	03/02/2015 23:48	STANDARD	2	2	MBF-6010	T1801010010	kg	5.0
3	03/02/2015 23:48	STANDARD	3	3	MBF-6010	T1801010010	kg	5.0
4	03/02/2015 23:48	STANDARD	4	4	MBF-6010	T1801010010	kg	5.0
5	08/02/2015 16:13	STANDARD	20230501	Andy	MBF-6010	T1801010010	kg	0.0

Data Collection Auto Manual

Model MBF-6010

S/N T1801010010

Date/Time 08/02/2015 16:13

Body Type STANDARD

Gender MALE

Age 23

Height 170.0 cm

Preset Tare 0.0 kg

Net Weight 70.8 kg

BMI 24.5

BF 11.6 %

FFM 62.6 kg

Export Collect Print Save